

Infiniti Comfort Compression Brace

Dual Handle Low Effort Pulley System

For Proper Fit Every Time:



- Center back panel on the spine.



- Wrap the left side to the front, positioning the front panel.



- Then wrap right side on top of the left panel to close and attach in front.



- Pull dual pulley handles out and away to desired compression and support.



- Place each handle on the fabric for a secure fit



- Adjust tension for sitting or standing.

For a consistent fit, make sure you follow these easy steps each time you apply the brace.



- Initially align pulley handle to the side of the brace prior to each fitting.



- Extend or pull the brace completely out prior to each application.

Care Instructions

- Hand wash in warm water, rinse well, towel and air dry completely.

To Shorten the Length of the Nylon Cord on the Pulley Handle:



1.

1. Detach the cord from the front clip
2. Pull the cord away from the 360 degree groove.



2.

3. Wrap the cord around the 360 degree side groove clockwise



3.



4.

4. Reattach the cord to the front clip

